

ORTONOM MEDICAL

Total Knee Prosthesis System Instruction for Use (IFU)

1. Implant Description

Ortonom has two different knee prosthesis models in primary knee system:

- OrtoKnee Fixed Total Knee System
- OrtoKnee Mobile Total Knee System.

Ortonom Total Knee Prosthesis System have same products for both of models. It is consisting of femoral component, tibial component, tibial insert and patellar component designed to be used together for total reconstructive prosthesis of the knee joint. All products in the Total Knee System are for single use.

a. Fixed Total Knee System

OrtoKnee Fixed Total Knee Prosthesis System can be used as Posterior Stabilized (P/S) and Cruciate Retaining (C/R). It is composed of Femur, Insert and Tibia components. If the doctor prefers Tibia Extension and Patella can be used.

Fixed feature ensures a constant locking of the Insert into the Tibia thanks to its special grooves in Tibia and Insert. Cruciate Retaining (C/R) Tibia components are metal knee components and are used with cement. There is no division between left and right.

b. Mobile Total Knee System

OrtoKnee Mobile Total Knee Prosthesis System can be used as Posterior Stabilized (P/S) and Cruciate Retaining (C/R). It is composed of Femur, Insert and Tibia components. If the doctor prefers Tibia Extension and Patella can be used.

Mobile feature offers the ability to make left and right rotation of Insert and Femur apart from Tibia. It has a special locking system to prevent the extrication of Insert from the Tibia.

Femoral Components

Femoral Components are metal knee components and are used with cement. They are designed with right and left separation and are available in posterior cruciate retaining (CR) and posterior cruciate sacrificing (PS) options.

It is made from CoCrMo Alloy raw material, which complies with the ISO 5832-4.

Systems consist of 5 sizes for Right and Left: C / D / E / F / G

Tibial Inserts

Tibial Inserts are polyethylene knee components and are used between the tibial and femoral components. There is no division between left and right. Posterior cruciate retaining (CR) and posterior cruciate sacrificing (PS) options are available.

It is made from UHMWPE raw material, which complies with the ISO 5834-2 standard.

Systems consist of 6 sizes. (1.5 / 2 / 3 / 4 / 5 / 5.5) There is no division between left and right.

Cruciate-Retaining (CR): It has a standard and no constrained bearing surface. Used with CR femoral component.

Cruciate Sacrificing/Posteriorly Stabilized (PS): It has more anterior/posterior constrained. Used with PS femoral component.

Tibial Components

Tibial Components are metal knee components and are used with cement.

They are available in fixed and mobile options. It is made from CoCrMo Alloy raw material, which complies with the ISO 5832-4. Systems consist of 6 sizes (1 / 2 / 3 / 4 / 5 / 5.5). There is no division between left and right.

Patellar Components

Patellar Components are polyethylene knee components and are used with cement. There is no division between left and right. The patellar components are cemented to the natural patella and move with the trochlear groove and condyles of the femoral component. It is used according to the surgeon's decision. It is made from UHMWPE raw material, which complies with the ISO 5834-2 standard. Systems consist of 5 sizes (26 / 30 / 34 / 38 / 42)

2. Purpose of Implant

The purpose of the implant is the reconstruction of degenerative knee joints. Total Knee Prosthesis System aims for pain relief and functional recovery of the knee joint. Degenerated joints are reconstructed appropriately for their anatomical structure with the aid of specialized surgical tools. For detailed information please refer to "Surgical Technique" section.

3. Indications

Joint replacement is indicated in patients with disabilities due to:

- Degenerative, posttraumatic or rheumatoid arthritis;
- Avascular necrosis of the femoral condyle;
- Posttraumatic loss of joint configuration,
- Especially patellofemoral erosion, dysfunction or previous patellectomy,
- Moderate valgus, varus or flexion deformities,
- Fractures that cannot be treated with other techniques.
- This device may also be used to correct previous surgical interventions that have failed. All devices are designed for cemented applications. Although total Knee Prosthesis are not intended to meet the loads or activity levels of normal healthy bone, they are a means of restoring motion and reducing pain for many patients.

4. Contraindications

Joint replacement is contraindicated in the presence of the following conditions:

- Infection (or a history of infection), acute or chronic, local or systemic;
- Inadequate bone quality that may affect implant stability;
- Muscular, neurological or vascular deficiencies that negatively affect the relevant extremity;
- Obesity;
- Alcoholism or other addictions;
- Sensitivity to materials;
- Loss of ligament structures;
- High levels of physical activity (e.g. competitive sports, heavy physical work).

5. Performance Design

Angular motion between the components of Total Knee Prosthesis System is designed to provide 0° - 130° movement. It is anticipated that the Total Knee Prosthesis System will experience a certain degree of abrasion due to the physical effects of daily activities on the knee joint. However, it is expected for this abrasion to stay at acceptable levels and for Total Knee Prosthesis System to remain serviceable for at least 15 years post implantation.

6. Side Effects

- Abrasion or breakage of the UHMWPE insert component,
- Tibial/femoral fracture,
- Temporary peroneal nerve palsy following surgical intervention,
- Partial or complete dislocation of the patella,
- Instability, position changes or aseptic loosening of components,
- Ligamentous laxity,
- Non-union of components,
- Infection,
- Narrow range of motion,
- Shortening of the extremities,
- Sensitivity to metal

7. Safe Configuration Criteria

Original femoral, tibial and insert components are required in order to assure Total Knee Prosthesis System's safe and secure implementation. Additionally, the operation should be performed with original Total Knee Prosthesis System surgical tools and as specified within the "Surgical Technique" document.

The dimensions specified in the table below can be used in conjunction with each other. The thickness of the insert component is not a determining parameter for this.

	FEMORAL COMPONENT					
	Size	C	D	E	F	G
FIX INSERT	1.5	X	X			
	2	X	X			
	3	X	X	X		
	4	X	X	X	X	X
	5			X	X	X
	5.5			X	X	X

	FIX TIBIA					
	Size	1	2	3	4	5 5.5
FIX INSERT	1.5	X				
	2		X			
	3			X		
	4			X	X	
	5				X	X
	5.5					X

	FEMORAL COMPONENT					
	Size	C	D	E	F	G
MOBILE INSERT	1,5	X	X			
	2	X	X	X		
	3	X	X	X		
	4		X	X	X	
	5			X	X	X
	5.5				X	X

	MOBILE TIBIA					
	Size	1	2	3	4	5 5.5
MOBILE INSERT	1,5	X				
	2		X	X		
	3		X	X		
	4				X	X
	5				X	X
	5.5					X

ORTONOM MEDICAL

Total Knee Prosthesis System Instruction for Use (IFU)

8. Expected Clinical Benefits

The total knee system, including both primary and revision knee prostheses, is intended to improve knee function and reduce pain by replacing the damaged knee joint articulation in skeletally mature patients.

9. Patient Population

Patients presenting with a clinically diagnosed necessity for a total Knee Prosthesis, compliant with the indications and not in conflict with the identified contraindications as defined in this packaging insert and who are musculoskeletally classified as adult.

No restrictions based on gender, ethnicity or stature (unless patient's physical condition would negatively impact the intended function or lifetime of the device). This device is not indicated for use with a paediatric population.

The patient should present with no mental health issues which would undermine either the rehabilitation phase or longevity of the device due to non-compliance with any limitations to function defined by the surgeon or the medical device manufacturer. Furthermore, the patient should be considered mentally capable to undertake the physical rehabilitation regime prescribed by the physician.

10. Adverse Effects

- Cardiovascular disorders including venous thrombosis, pulmonary embolism and myocardial infarct can occur.
- Damaging of blood vessels or hematomas can be seen.
- Delayed wound healing: Deep wound infections (early or late) that might require the removal of the prosthesis can occur. Arthrodesis of the relevant joint or amputation of the extremity can be required in rare cases.
- Osteolysis (progressive bone lysis). Osteolysis can be asymptomatic, and therefore, periodic radiographic examinations are vitally important to prevent any possible severe complications in the future.
- Unwanted lengthening or shortening of the extremity can occur.
- Fatigue fractures of the components of the prosthesis can arise from trauma, exhausting physical activity, misalignment, defective implantation, service period, loss of fixation, nonunion or overloading.
- As a potential result of surgical trauma resulting in pain or numbness in the affected extremity, temporary or permanent nerve damage, peripheral neuropathies or subclinical nerve damages can be seen.
- Sensitivity against the metal or histological or allergic reactions or adverse foreign material reaction can occur as a result of implantation of a foreign material.
- Macrophages against the implant or tissue reactions containing foreign material reactions can be seen around the implant.
- Embolism can occur in patients who have extreme sedentary lifestyle.

11. Warnings

- There may be patient sensitivity or allergic reaction to implant materials, particularly to metal ions. The material type of each implant is indicated on the product box label. As part of preoperative planning, the physician should assess potential risks related to sensitivity or allergy to implant materials and inform the patient accordingly.
- Do NOT resterilize the Total Knee Prosthesis System under any circumstances.
- Do NOT use implants if sterility may have been compromised.
- Before surgery, the product packaging should be inspected for tears, damage, or any compromise of the sterile barrier. If the sterile barrier is compromised, the product must not be used and should be returned to Ortonom.
- Check the product label to verify the expiration date. Do not use the product after its expiration date.
- When unpacking the implant, verify the reference number and dimensions on the product box label and the inner labels.
- The Total Knee Prosthesis System is for single use only. Used products must not be reused.
- Do not use prosthetic components from other knee systems with Ortonom knee components, as incompatible sizing and bearing surfaces may result in premature wear, malalignment, and failure.
- Before use, inspect each implant to ensure that there is no visible damage.
- Care should be taken to protect polished surfaces, especially convex and concave surfaces, from nicks and scratches. Contact with polished surfaces should be avoided.
- Do not expose the Total Knee Prosthesis System to chemicals in the clinical environment. Do not clean the implant with alcohol or subject it to similar processes. Overweight patients may experience decreased implant performance.
- The prosthesis cannot fully replace normal healthy bone. The patient should be informed that the prosthesis may fracture or become damaged as a result of certain movements, trauma, strenuous activity, malalignment, incomplete implant placement, loss of fixation, non-union, long-term use, or excessive body weight. The expected service life of the prosthesis is limited, and replacement may be required in the future.
- If the Total Knee Prosthesis System needs to be removed, refer to the Surgical Technique document.
- If the Total Knee Prosthesis System needs to be disposed of, it should be considered medical waste and handled accordingly.
- Products of different manufacturers must not be used together.

Components that can be used together according to EN ISO 21534 Annex-C:

- CoCrMo Alloy (ISO 5832-4)/ Titanium Alloy (ISO 5832-3)
- CoCrMo Alloy (ISO 5832-4) / CoCrMo Alloy (ISO 5832-4)
- Implant Steel (ISO 5832-1)/ Titanium Alloy (ISO 5832-3)
- Implant Steel (ISO 5832-1)/ Implant Steel (ISO 5832-1)

Components that are incompatible with each other:

- Implant Steel (ISO 5832-1)/ CoCrMo Alloy (ISO 5832-4)

12. Residual Risks

- Wear and tear on the components can lead to increased loosening and bone damage. This may require revision surgery.
- Wear and tear of the components can lead to increased loosening and damage to the bone. This may require revision surgery.
- Due to trauma or excessive loading—particularly if the bone tissue is weak at the time of implantation—a hole may form or a fracture may occur in the femur or acetabulum.
- Cardiovascular disorders, including venous thrombosis, pulmonary embolism, or myocardial infarction.
- Fatigue fractures in the prosthetic component may occur as a result of trauma, strenuous activity, misalignment, improper implant placement, duration of use, loss of fixation, nonunion, or excessive weight.
- Patients should be advised that prostheses cannot replace normal, healthy bone; that prostheses may fracture or pose a risk as a result of certain movements or trauma; that their expected service life is limited; and that replacement of the prosthesis may be necessary in the future.
- Products are supplied sterile. Re-sterilization of the products is not recommended. Please report any products that have lost their sterility to the manufacturer.
- Check the expiration date. If you encounter a product past its expiration date, please contact the manufacturer.
- The devices are for single use only. Do not reuse a used product. This rule is clearly stated.

13. Removal Of the Implant

The period of the implant within the patient's body can change according to the daily activity of the patient. The expected product life of the implant is projected to last for the patient's lifetime, with a minimum of 15 years. Implants can be removed according to the surgeon's decision.

14. Sterilization

Implants are submitted as sterile. Sterilization method will be shown on the packaging. All implants sterilized by gamma irradiation are exposed to a minimum of 25 kGy and a maximum of 45 kGy of gamma radiation. Sterile products will remain so unless the integrity of the package is broken. The products are presented as sterile. Re-sterilization of the product is not recommended.

Please inform the manufacturer about any products with impaired sterility although offered as sterile.

15. Shelf Life

The shelf life for Total Knee Prostheses sterilized with Gamma sterilization is 5 years.

ORTONOM MEDICAL

Total Knee Prosthesis System Instruction for Use (IFU)



Please check the expiry date. Please contact the manufacturer for products expired.

16. Storage And Maintenance Conditions

Implants must be maintained in a dry and clean environment and protected from direct sunlight, excessive heat and humidity.

17. Severe Events

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

18. MRI Compatibility

The Total Knee Prosthesis has not been evaluated for safety in the MR environment. It has not been tested for heating or unwanted movement in the MR environment. The safety of Total Knee Prosthesis in the MR environment is unknown. Performing an MR exam on a person who has this medical device may result in injury or device malfunction. Ortonom products have not been evaluated for safety and compatibility in the MRI environment.

19. Disposal Implant

Please apply disposal procedure for implants accordance with hospital rules.

20. Product Description and Implant Materials

Component	Fixation Method	Material	ASTM Standard	ISO Standard
Femoral Component	Fixated to the distal femur with cement	CoCrMo Alloy	ASTM F75	ISO 5832-4
Tibial Component	Fixated to the proximal tibia with cement	CoCrMo Alloy	ASTM F75	ISO 5832-4
Insert Component	Fixed by fitting it into appropriate tibial component. This design, which restricts movement in the AP and ML axes, does not restrict movement in the rotation axis	Ultra-High Molecular Weight Cross Linked Polyethylene (UHMWPE)	ASTM F648	ISO 5834-2
Patellar	The conical protrusion design is fixed by fitting it into the proximal conical hole of the tibial plateau. This design, which restricts movement in the AP and ML axes, does not restrict movement in the rotation axis	Ultra-High Molecular Weight Cross Linked Polyethylene (UHMWPE)	ASTM F648	ISO 5834-2

Stem	Used to reinforce intramedullary fixation of the tibia from proximal to distal	Ti6Al4V Titanium Alloy	ASTM F136	ISO 5832-3
------	--	------------------------	-----------	------------

21. Raw Materials Information

CoCrMo raw material is produced in accordance with ASTM F75 and ISO 5832-4 standard. Raw material content is given in the table below.

Element	Composition, % ISO 5832-4	Composition, % ASTM F75
Chromium (Cr)	26,5 - 30,0	27,0 - 30,0
Molybdenum (Mo)	4,5 - 7,0	5,0 - 7,0
Iron (Fe)	1,0 max.	0,75 max.
Manganese (Mn)	1,0 max.	1,0 max.
Silicon (Si)	1,0 max.	1,0 max.
Carbon (C)	0,35 max.	0,35 max.
Nickel (Ni)	1,0 max.	0,50 max.
Nitrogen (N)	0,25 max.	0,25 max.
Cobalt (Co)*	Balance	Balance
Tungsten	-	0,20 max.
Phosphorous	-	0,020 max.
Sulfur	-	0,010 max.
Aluminum	-	0,10 max.
Titanium	-	0,10 max.
Boron	-	0,010 max.

UHMWPE raw material is produced in accordance with ISO 5834-2 standard. Raw material content is given in the table below.

Element	Maximum Quantity Permitted (mg/kg)
Ash	125
Titanium	40
Calcium	5
Chlorine	30
Aluminium	20

Ti6Al4V ELI raw material is produced in accordance with ISO 5832-3 standard. Raw material content is given in the table below.

Element	Composition, %
Nitrogen (N), max	0.05
Carbon (C), max	0.08
Hydrogen (H), max	0.012
Iron (Fe), max	0.25

Oxygen (O), max	0.13
Aluminum (Al)	5.5-6.50
Vanadium (V)	3.5-4.5
Titanium (Ti)	Residual

22. CMR Description

RS Identifier	RS Description	Risk
	Cobalt: CAS No. 7440-48-4 EC No. 231-158-0	This device or one or more of its components contain the substance defined as CMR 1B at a concentration of more than 0.1% by weight . Existing scientific evidence and biocompatibility studies support that medical device made from cobalt alloys or stainless-steel alloys do not increase the risk of cancer or cause adverse effects on reproduction .

23. Preoperative Planning and Postoperative Care

**See Surgical Technique document.

24. Symbols Shown on the Label

SYMBOL DESCRIPTION		
	European Medical Device Directive 93/42/EEC (as amended by Directive 2007/47/EC), Article 17 European Medical Device Regulation 2017/745, Article 20	CE Symbol
1984	European Medical Device Directive 93/42/EEC (as amended by Directive 2007/47/EC), Article 17 European Medical Device Regulation 2017/745, Article 20	NB Number
	ISO 15223-1 (Section 5.7.7*)	Medical device
	ISO 15223-1 (Section 5.1.5) ISO 7000 (Symbol No. 2492)	Batch code
	ISO 15223-1 (Section 5.1.6) ISO 7000 (Symbol No. 2493)	Catalogue number
	ISO 15223-1 (Section 5.7.10*)	Unique Device Identifier

ORTONOM MEDICAL

Total Knee Prosthesis System Instruction for Use (IFU)

	ISO 15223-1 (Section 5.4.3) ISO 7000 (Symbol No. 1641) ISO 15223-1 Annex A / A.15	Consult instructions for use
	ISO 15223-1 (Section 5.1.1) ISO 7000 (Symbol No. 3082)	Manufacturer
	ISO 15223-1 (Section 5.1.11) ISO 7000 (Symbol No. 2497)	Country of manufacture
	ISO 15223-1 (Section 5.1.4) ISO 7000 (Symbol No. 2607)	Use-by date
	ISO 15223-1 (Section 5.2.12*) ISO 7000 (Symbol No. 3704)	Double sterile barrier system
	ISO 15223-1 (Section 5.2.4) ISO 7000 (Symbol No. 2502)	Sterilized using irradiation
	ISO 15223-1 (Section 5.4.2) ISO 7000 (Symbol No. 1051)	Do not re-use
	ISO 15223-1 (Section 5.2.6) ISO 7000 (Symbol No. 2608)	Do not resterilize
	ISO 15223-1 (Section 5.2.8) ISO 7000 (Symbol No. 2606)	Do not use if package is damaged and consult instructions for use
	ISO 15223-1 (Section 5.3.2) ISO 7000 (Symbol No. 0624)	Keep away from sunlight
	ISO 15223-1 (Section 5.4.4) ISO 7000 (Symbol No. 0434A)	Caution
	ISO 15223-1 (Section 5.4.10)	Contains hazardous substances

Information supplied with the device

The Summary of Safety and Clinical Performance (SSCP) is available in the European Database on Medical Devices (Eudamed, <https://ec.europa.eu/tools/eudamed>) or you can it via the manufacturer's website: www.ortonom.com.tr.

e-IFU

You can access the e-IFU at <https://ortonom.com.tr/qrdirected-010> Ortonom + Hip and Knee Implant Systems . You can also access the e-IFU via the QR code on the label and box. If you cannot access the e-IFU, please contact the manufacturer.

Implant Card

The implant card is physically located inside the product box within the device packaging. The implant card is contained in the Level 2 package inside the box. If you are unable to locate the physical implant card, please contact Ortonom Medikal.

Manufacturer



Ortonom Medikal Ürünleri San. Tic. Ltd. Şti.
Address: Çamlıca Mah. Anadolu Bulv. Timko Ticaret Merkezi No:20/M-6
Yenimahalle/ Ankara/ Türkiye
Tel.: +90 (312) 440-8151
Fax.: +90 (312) 440-8586
E-mail: info@ortonom.com.tr
Web Site: www.ortonom.com.tr

